

## MiniBox+ Cleaning and Disinfection Instructions



**Warning:** Before performing Full Cleaning or Disinfection procedures, be sure to turn the power off and disconnect the power cord from the device



**Caution:** Avoid contact of any liquids with the internal system parts. There is no protection of the electronics against incoming water or liquids. Pay extra care when cleaning around the slots found at the side of the device.



**Note:** We recommend the use of Oxivir Tb spray (EPA Reg. No 70627-56) and Oxivir wipes (EPA Reg. No 70627-60) for both cleaning and disinfection of the MiniBox™.



**Note:** Do not clean the system with abrasive cleaning or disinfectant solvents that could scratch or damage the system.



**Note:** Cleaning must be completed before disinfection. If the device does not appear visually clean at the end of the cleaning procedure, the above cleaning instructions must be repeated.

### Simple Cleaning

**Clean the MiniBox™ after each patient as follows:**

1. Wipe the outer surfaces of the hand held unit using Oxivir wipes.
2. Allow the surface to dry completely.

### Full Cleaning

**Thoroughly clean the MiniBox™ when you suspect it to be contaminated, as follows:**

1. Apply Oxivir Tb spray to all the outside surfaces of the device including the outer surface of the hand held unit and of the main body of the device.
2. Allow the surfaces to remain wet for one minute.
3. Wipe the surfaces dry using a soft cloth or a disposable wipe.